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	Caution
	Consult instructions for use
	Keep bottles out of direct sunlight
	Temperature Limitation
	Single patient - multiple use
	Sterilized using aseptic processing techniques
	Do not resterilize. Resterilization may reduce the effectiveness of the product
	Do not use if package is damaged
	Keep out of reach of children
	19G needle (application pressure of ~ 10 psi)
	Medical device
	Batch code / LOT #
	Date of manufacture Country of manufacture
	Use by date
	Catalog number
	Manufacturer
	Distributor
	UDI - Unique device identifier

CAUTIONS AND WARNINGS

- For external use only. Avoid contact with eyes.
- Use only undamaged bottles.
- Do not use in patients with a history of hypersensitivity to any of the ingredients.
- Discontinue use if irritation develops. Avoid usage of the turbid or colored solution.
- Protect bottle from direct contact with the wound.
- Bottles that have come into contact with the wound must be discarded after use.
- For single patient use only.
- If the bottle is closed immediately after use and the top of the bottle is protected from contamination, the solution may be reused within 28 days from the first opening.
- Store between 10°C-30°C (50°F-86°F) in a dry place.
- The empty bottle can be disposed to the regular trash can. A bottle that contains a solution should be cleared off to an appropriate site.
- Do not apply more than 700mL of SilverStream solution within 24 hours.

SYMBOLS

The image above provides a description of the graphic symbols that appear on the SilverStream® label.

English (EN)

INTENDED USE AND INDICATIONS FOR USE

SilverStream® is intended for use under the supervision of a healthcare professional for mechanical cleansing and removal of foreign material including micro-organisms and debris from wounds such as stage I-IV pressure ulcers, stasis ulcers, diabetic foot ulcers, post-surgical wounds, first and second-degree burns, cuts abrasions and minor skin irritations.

GENERAL DIRECTIONS FOR USE

SilverStream® is designed for management and moisturizing of wounds.

Facilitate wound healing by applying the SilverStream® solution directly from the bottle on the wound in one of the 4 ways:

- (1) Applying by means of a syringe or bulb.
- (2) Applying with device designed to allow for irrigation or lavage.
- (3) Spraying onto the wound using a spray bottle.
- (4) Pouring onto a gauze/dressing or sponge.

Apply 1-2 times daily, according to Medical Practitioner's instruction.

Before removal of the gauze/dressing or sponge, moisten with SilverStream® solution for easier removal.

Between dressing changes, wash the wound with water or soap and water.

DO NOT INJECT

Note: The wound may be cleansed if SilverStream® is applied with a 19G needle (application pressure of ~ 10 psi).

DO NOT wash the wound with Saline, Before or After using SilverStream®.

INGREDIENTS

Ingredients include WFI (water for injection), glycerol (glycerin), sorbitan monolaurate (Tween-20), TRIS, menthol, and silver nitrate 0.01% (w/v).

Each mL contains ionic silver in the form of silver nitrate 0.01% (w/v).

PACKING

Each package contains twelve (12) bottles of 250mL or twenty-four (24) bottles of 100mL or twelve (12) bottles of 500mL and instructions for use leaflet, which is part of the bottle label.